### DESCRIPTION OF PRODUCT:

Posey Standard and Incontinent Brief Hipsters®.

Posey Hipsters® feature impact absorbing, soft foam pads over the critical fracture area to help reduce the risk of injuries from a fall, such as hip fractures. Hipsters® Innerwear is made of a comfortable 95% cotton/5% Spandex blend with a low profile pad over each hip. Hipsters® Outerwear is made of a high quality blend of pre-shrunk 7.8 oz, 50/50 cotton/polyester.

### INTENDED USE:

- Hipsters® are intended to reduce the risk of patient hip injury due to a fall.
- Follow your facility's policies and guidelines for frequency of patient monitoring and skin assessment.

### CONTRAINDICATIONS:

None.

### APPLICATION INSTRUCTIONS:

#### Standard and Incontinent Brief

With the Posey label in the back, step into the garment and gently slide it up over the hips. Adjust to assure that the foam pads are properly aligned with and cover the hip joint.

### PROPER PAD PLACEMENT:

Select the proper size (see chart below) and have the patient try on the Hipsters for proper fit. Make sure that Hipster pads cover the hip joint (greater trochanter).

### TO DO THIS:

1. Have the patient stand up with their arms hanging down by their side.
2. The wrist joints should align with the center of the pad on each side.
3. Press your fingers under one pad and have the patient lift the heel on that side.
4. You should be able to feel the hip joint moving under the pad.
5. Repeat on the other side. If the pads do not cover the hip joints on each side, you must re-assess the fit.

Wrists approximate trochanter

Verify that trochanter is under pad

Pad should cover trochanter area

### SIZING CHART

Size	Waist Measurement	Hip Measurement
XS	26"-28" or 66-71 cm	33"-35" or 83-88 cm
S	28"-30" or 71-76 cm	35"-37" or 88-93 cm
M	30"-34" or 76-86 cm	37"-41" or 93-104 cm
L	34"-38" or 86-96 cm	41"-45" or 104-114 cm
XL	38"-42" or 96-106 cm	45"-49" or 114-124 cm
XXL	42"-46" or 106-116 cm	49"-53" or 124-134 cm

If waist measurement is larger than the hip measurement, select the size that corresponds to the larger waist measurement.

Standard Unisex Brief easily fits over undergarments, or can be worn as underwear.

**REF 6016** Hipsters, Standard Brief

Incontinent Brief features a snap front for easier application over adult diapers.

**REF 6017** Hipsters, Incontinent Brief with Removable Pads

### LAUNDERING INSTRUCTIONS:

All Posey Hipsters can be washed per CDC guidelines.

Wash Hipsters at 120°F (50°C) and dry at medium temperature.

- Attach hook and loop straps to prevent lint build-up on hook during laundry cycle. If hook and loop does not adhere due to lint, clean hook material with a stiff brush.

The complete Guidelines for Laundry in Health Care Facilities are available at: [www.cdc.gov/od/ohs/biosfty/laundry.htm](http://www.cdc.gov/od/ohs/biosfty/laundry.htm). The CDC states that "the risk of actual disease transmission from soiled linen is negligible" and "recent studies have shown that a satisfactory reduction of microbial contamination can be achieved at lower water temperatures of 72°-122°F (22°-50°C) when the cycling of the washer, the wash formula, and the amount of chlorine bleach are carefully monitored and controlled."

The CDC offers the following advice for home care: "In the home, normal washing and drying cycles including 'hot' or 'cold' cycles are adequate to ensure patient safety. Manufacturers instructions for the machine and the detergent or wash additive should be followed closely." See: [http://www.cdc.gov/ncidod/dhqp/bp\\_laundry.html](http://www.cdc.gov/ncidod/dhqp/bp_laundry.html).

### WARNING: CHECK FOAM PADS AFTER EACH WASHING.

- Remove the pads from the Hipster pocket and inspect for damage prior to use. Replace cracked, broken, or damaged pads or pouches, or discard the garment. Pads are sealed in a waterproof pouch to protect them from water or urine. If a pouch is cut or the seal is broken, moisture may enter pouch and waterlog the foam. This may promote bacterial growth and will reduce protection from a fall.
- For models with non-removable pads, squeeze the pads and feel for any evidence of cracked or broken pads and for any evidence pouch damage or moisture. If one or both of the pads are damaged, discontinue use and discard the garment.
- Hipsters are Temperature-Responsive. The Posey Hipster Pads are "temperature-responsive." Right out of the package, or off the shelf, Hipsters may feel "stiff". However, body heat causes the foam to soften and conform to the body shape.

If a patient falls with the Hipster on, replace the garment or the removable pads before patient use.

Due to the random nature of falls, TIDI Products, LLC., cannot guarantee that hip trauma will not occur. Check patient's skin often. Wash Hipsters after each incontinent episode to reduce the risk of skin breakdown.

**STORAGE AND HANDLING:** This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

**WARNING:** Properly dispose of the product per facility's policy for **BIOHAZARDOUS** materials.

EN

Report any serious incident related to device use to TIDI Products and the Member State Competent Authority. Translations of this document in French, Spanish, Dutch, German, Italian, and Portuguese are located at [www.tidiproducts.com/IFU](http://www.tidiproducts.com/IFU).

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© Posey Products, LLC. Tutti i diritti riservati.