# Posey® Foam Tracheostomy Ties – 8197S, 8197M, 8197L, 8197XL

**DESCRIPTION OF PRODUCT:** The Posey Foam Tracheostomy Ties are non-sterile and are wrapped around the patient's head and neck area to hold tracheostomy tubes and oxygen cannula in place.

**INTENDED USE:**

The Posey Foam Trach Tie is used to hold a tracheal tube or oxygen cannula in place.

**CONTRAINDICATIONS:** None.

Fig. 1

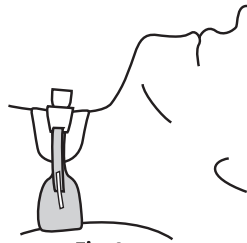
Fig. 2

**APPLICATION INSTRUCTIONS FOR TRACHEAL TUBE:**

1. Thread the “hook” end of the tie through one slot of the tracheostomy tube and attach the “hook” fastener to the fuzzy side of the neck pad or strap. Make sure entire fastener is attached as shown in Fig. 2. Slot should be at least 3/8” wide to allow for easy insertion (Fig. 1).
2. Bring the foam neck collar behind the patient's neck and thread the other “hook” end through the other slot in the tracheostomy tube.
3. Adjust to fit. A proper fit is achieved when the entire tie rests smoothly on the neck with no slack. Refer to sizing table for appropriate size Trach Tie for your patient's neck size. DO NOT attach too tightly. Foam should not be compressed when applied.
4. Secure “hook” fastener to fuzzy side of neck strap or neck collar (Fig. 2). Make sure entire fastener is attached as shown in Fig. 2 on both sides.
5. If desired, the neck strap may be knotted to shorten.

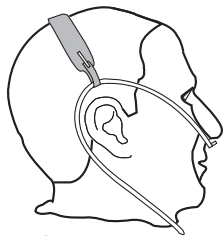
Fig. 3  
Top of Head

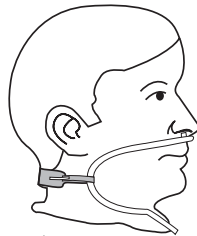
Fig. 4  
Back of Neck

**APPLICATION INSTRUCTIONS FOR OXYGEN CANNULA SUPPORT:**

1. Position the oxygen cannula as shown in Fig. 3 or Fig. 4.
2. Secure one side of the tie to the oxygen cannula. Make sure entire fastener is attached as shown in Fig. 3 or Fig. 4.
3. Bring the foam neck collar behind the patient's neck or over the patient's head. Secure the other side of the tie to the other side of the oxygen cannula, taking care not to displace the oxygen cannula.
4. Adjust to fit. A proper fit is achieved when the entire tie rests smoothly on the neck or head with no slack. DO NOT attach too tightly. Foam should not be compressed when applied.
5. Secure “hook” fastener to fuzzy side of the tie. Make sure entire fastener is attached as shown in Fig. 3 or Fig. 4 on both sides.
6. If desired, the neck strap may be knotted to shorten.

**POSEY FOAM TRACHEOSTOMY TIES:** Available in four sizes  
(One dozen per box)

<b>REF 8197S</b>	Small, 9”L x 1”W (23 cm x 3 cm), fits necks 7-9” (18-23 cm)
<b>REF 8197M</b>	Medium, 18 ½”L x 1”W (47 cm x 3 cm), fits necks 9-14” (23-35 cm)
<b>REF 8197L</b>	Large, 23”L x 1”W (60 cm x 3 cm), fits necks 13-19” (33-48 cm)
<b>REF 8197XL</b>	X-Large, 30”L x 1”W (70 cm x 3 cm), fits necks 16-24” (41-61 cm)

**WARNING**

- **MONITORING:** Be sure to follow your facility's policies and guidelines for frequency of patient monitoring, for proper product application, and skin integrity. ALWAYS monitor patients carefully to ensure they are not at risk for disrupting lifesaving therapy. Constant supervision, electronic monitoring and/or the use of a restraint may be required in conjunction with proper medical supervision per your facility's policies.
- **INSPECTION:** Inspect before each use; check for broken stitches or parts, or torn, cut or frayed material. DO NOT use soiled or damaged product.
- **REMOVAL:** Do not force tabs through the slot when removing trach tie. This could cause accidental extubation or displacement of the tube. If the slot is too small to allow for easy removal, cut off the hook and loop tab, and pull the tie gently.
- Discard if “hook” does not firmly attach to fuzzy backing.
- Dispose of the product per facility's policy if worn or damaged.

EN

Report any serious incident related to device use to TIDI Products and the Member State Competent Authority. Translations of this document in French, Spanish, Dutch, German, Italian, and Portuguese are located at [www.tidiproducts.com/IFU](http://www.tidiproducts.com/IFU).

TIDI Products, LLC • 570 Enterprise Drive, Neenah, WI 54956 USA  
Phone: 1.800.447.6739 • International: +1.920.751.4036 • [www.tidiproducts.com](http://www.tidiproducts.com)

© Posey Products, LLC. All rights reserved.

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UK Responsible Person  
Emergo Consulting (UK) Limited  
c/o Cr360 – UL International  
Compass House, Vision Park Histon  
Cambridge CB249BZ  
United Kingdom

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TIDI Products, LLC • 570 Enterprise Drive, Neenah, WI 54956 EUA
© Posey Products, LLC. Todos os direitos reservados.