


# PenBlade® Instructions for Use

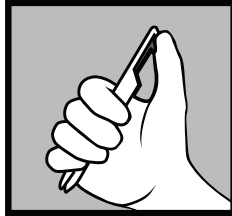
One-handed use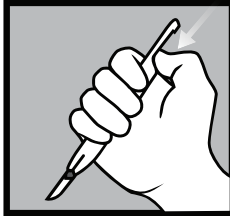


Pen-style activation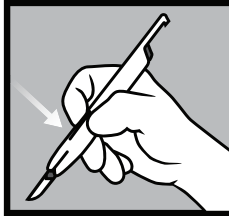

Press-release retraction

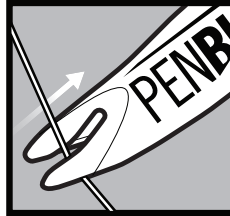Safe-trimming groove

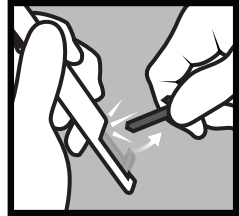Disable (Optional)

REF

PB-M-10  
PB-M-11P  
PB-M-15

STERILE/EO

TIDI Products, LLC  
570 Enterprise Drive  
Neenah, WI 54956 USA  
800.521.1314  
+1 920.751.4300

MD

[www.tidiproducts.com/IFU](http://www.tidiproducts.com/IFU)

**INTENDED USE:** The PenBlade® Safety Scalpel is intended for the incision of human skin or tissue or the cutting of sutures and catheters up to 6Fr.

**DEVICE DESCRIPTION:** The PenBlade® Safety Scalpel is a scalpel which contains a mechanism to extend and retract a blade between an exposed and a retracted safe position. The PenBlade is available in three scalpel sizes; 10, 11P, or 15 blades.


- The device is intended for use for all patient populations.
- The device is suitable for use upon removal from packaging. No set-up is required.
- Ruler markings on the device are for reference measurements only.

#### PRECAUTIONS:


- The device is a sterile, single-use only product. Do not re-sterilize and/or reuse; injury or infection may occur.
- The scalpel blade is sharp and can cause harm if improperly used.
- Do not use the device if damage to the device or device sterile-barrier packaging is observed or suspected.
- If the extension button is jammed, do not force the scalpel blade to extend.
- Report any serious incident related to device use to TIDI Products and the Member State competent authority.
- Do not autoclave or expose the device to organic solvents.
- Use the device before the "Use By" date noted on the packaging.
- Do not cover the scalpel blade tip with hand or other part of body at any time.
- Always retract the scalpel blade to the safe position when not in use.
- After use, retract the blade to the safe position and dispose of the device in a sharps/biohazard container. Failure to dispose of the device properly after use may result in a sharps injury and/or infection.


# PenBlade® Instrucciones de uso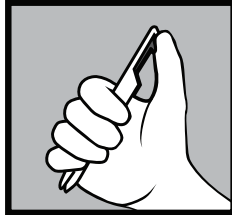


Uso con una sola mano

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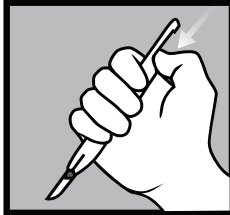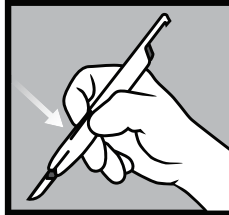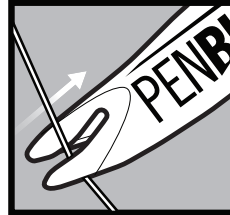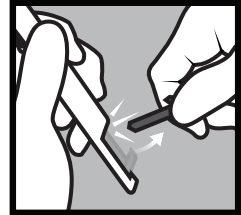
REF

PB-M-10  
PB-M-11P  
PB-M-15

STERILE EO

TIDI Products, LLC  
570 Enterprise Drive  
Neenah, WI 54956 USA  
800.521.1314  
+1 920.751.4300

MD

[www.tidiproducts.com/IFU](http://www.tidiproducts.com/IFU)

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