
# Grip-Lok®

**REF** 3309MCS-TA-2

- EN** Catheter Fixation for ARROW® PICCMidline
- DE** Katheter-Fixierung für ARROW® PICCMidline
- FR** Dispositif de fixation de cathéter pour ARROW® PICCMidline
- IT** Dispositivo di fissaggio per catetere ARROW® PICCMidline
- NL** Katheterbevestiging voor ARROW® PICCMidline
- SV** Kateterfästning till ARROW® PICCMidline
- ES** Fijación de catéter para ARROW® de ARROW®
- PT** Fixação do Cateter para ARROW® PICCMidline
- EL** Στερέωση καθετήρα για τις προετοιμασμένες στο χώρο κεντρικές καθετήρες (PICCY) καθετήρα με τους γραμμές ARROW®
- FI** Katetrin kiinnitys ARROW® PICCMidline
- DA** Kateterfikseing til ARROW® PICCMidline
- S** Holleghesfesting fyrir ARROW® PICCMidline
- NO** Fikseing av kateter for ARROW® PICCMidline
- PL** Mocownia cewnika do ARROW® PICCMidline
- CS** Zajímaví katétru pro ARROW® PICCMidline
- SK** Fixácia katétra pre ARROW® PICCMidline

44U00016 REV 01

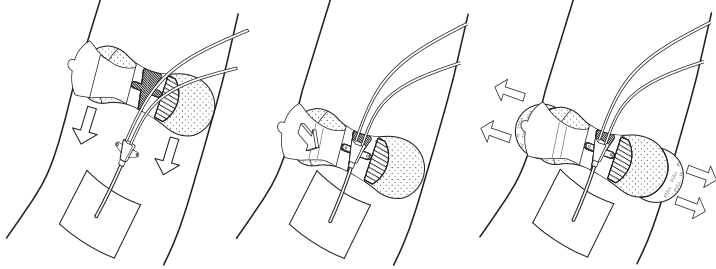## ANWEISUNGEN

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# Teleflex®

TIDI Products, LLC  
570 Enterprise Drive  
Neenah, WI 54956, USA  
Phone: + 1 920 751 4300  
[tidiproducts.com/securement](http://tidiproducts.com/securement)

2797

UK  
CA  
0086

UK Responsible Person:  
Emergo Consulting (UK) Limited  
c/o Cr 360 - UL International  
Compass House, Vision Park, Histon  
Cambridge CB24 9BZ  
United Kingdom

Grip-Lok is a registered trademark of TIDI Products, LLC.  
May be patented: see [www.tidiproducts.com/virtual-patent-marking](http://www.tidiproducts.com/virtual-patent-marking)

2023-08-09

## INSTRUCTIONS

**EN**

English

Intended for use in the general population, for adhesive securement of the catheter hub, tube or line to the patient's skin.

- Select the area for the placement of the Grip-Lok. Note: The catheter should be already inserted in the patient.
- Prepare the skin according to the standard hospital protocol for dressing application. Hair removal may be required on some patients for better adhesion.
- Open the top flap, remove the interior liner and hold the flap open.
- Slide the Grip-Lok under the catheter and adhere in the exposed adhesive cut-out space provided.
- Secure the top flap over the catheter.
- Hold the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.

**Direction:** For additional skin protection, use skin prep pad prior to application of stabilization device.

**Note:** Use of an alcohol swab may help removal of securement device from the skin. If you experience severe redness, itching, swelling, or irritation of the skin, consult your physician as this may be a sign of an allergic reaction.

Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections.

Replace securement device if soiled or saturated in fluid or if device shows signs of wear or damage.

**Note:** If being used in a medical facility, dispose according to medical facility and municipal guidelines.

Do not use if the product's sterile barrier system or its packaging is compromised.

Report any serious incident related to device use to TIDI Products and the Member State competent authority.

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Grip-Lok es una marca registrada de TIDI Products, LLC.